

# FlexiShoe™


Completely adjustable for pressure free comfort

Weighing less than six ounces each, the pressure free **FlexiShoes** are constructed of Burr-Lok, a soft, washable, breathable material.

The **FlexiShoes** are adjustable in length and width and will accommodate most foot configurations.

The **FlexiShoe** soles extend over the heel of the foot for extra comfort and protection.

In sizes small, medium, large and extra large, styled for both men and women, the Flexishoe features a neutral toe which will permit a FlexiShoe to be worn on either the left or right foot.

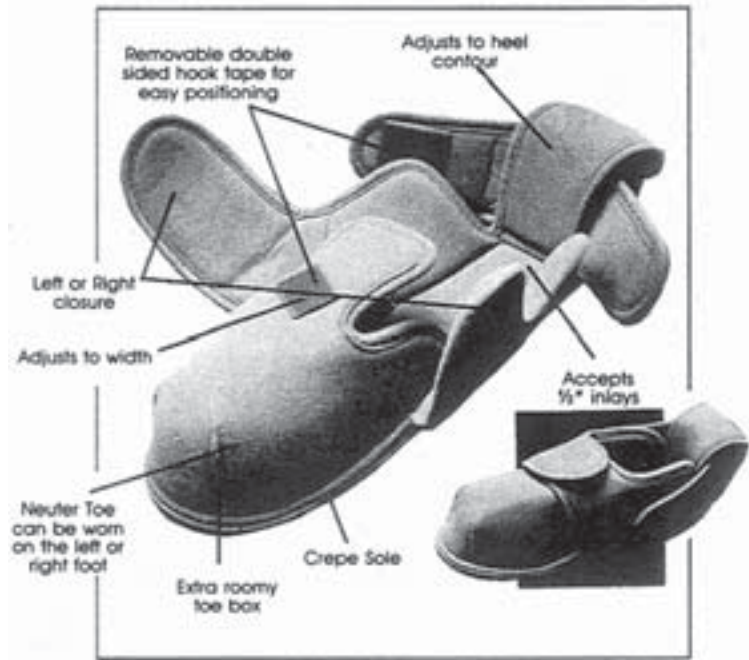

## Instructions

For a left or right closure, place the removable double-sided hook tape on the flap of your choice. Adjust for width. Position the other flap to cover the tape.

To adjust the heel width, remove the double sided hook tape (as illustrated) from the left and right ends of the heel tab.

### For narrow heels:

Place the hook tapes on the inner surface of each FlexiShoe. Position the heel tab to cover the tapes.

### For wide heels:

Place the hook tapes on the outer surface of each Flexishoe. Position the heel tab to cover the tapes.

Size	Fits Shoe Sizes
Small	Women's 6 to 9 Men's 5 to 8
Medium	Women's 9 to 12 Men's 8 to 11
Large	Men's 11 to 14 Width C to EE
X-Large	Men's 14 to 16 Width C to EEE

## Washing Instructions

Remove the hook tape strips. Hand launder your **FlexiShoes** in warm water with mild soap. Air dry at room temperature. **FlexiShoes** are not manufactured for outdoor wear

Made in the U.S.A.

U.S. Patents 3,762,075 and 4,136,468; Canadian Patent 1,092,348; UK Patent GB2014431B



Vittoria Phoenix inc  
1534 Old Brock Street  
Vittoria, ON N0E 1W0

[www.orthoVP.com](http://www.orthoVP.com) Tel: (800) 563 - 1296  
[vphoenix@on.aibn.com](mailto:vphoenix@on.aibn.com) Fax: (888) VP FAX IT



Type	Sizes
FootRest	Women's Sm, Med, Lge Men's Sm, Med, Lge
Wocker	Women's 9 to 12 Men's 8 to 11

**Same Last as Flexi-Shoes on previous page**

The **WockerShoe** and **FootRest** surgical shoes are designed to ease post-operative and post-traumatic conditions of the foot. The combination of Burr-Lok fabric with the removable hook tapes on the **FootRest** and removable elastic band on the **WockerShoe** enhances the shoe's ability to accommodate bulky dressings at the forefoot and rear-foot.

Easily manageable in adjustability and ease of application, you and your patients will appreciate the craftsmanship, style and comfort.

The highly adjustable rigid plastic sole shoes may be utilized in the treatment of fractures, dislocations, bruises and sprains; post operative care of metatarsal osteotomies, exostectomies, bunionectomies and other surgical procedures.